

neodisher® MediZym

Liquid, pH-neutral cleaning agent
for use in special washing machines



| | |
|---|--|
| Main fields of application: | Mechanical cleaning of surgical instruments, anaesthesia equipment, flexible endoscopes, containers and other medical utensils. |
| Characteristics: | <p>neodisher MediZym is a particularly environmentally-friendly pH-neutral cleaning agent with enzymes and other cleaning adjuvants. Dried and denatured blood residues are removed by neodisher MediZym while being extremely kind to equipment.</p> <p>Stainless steel, instrument steel, light metals, glass, the usual plastics and the materials of anaesthesia equipment and flexible endoscopes are resistant to working solutions of neodisher MediZym.</p> |
| Application and dosage: | <p>For machine cleaning of instruments: 0.2 - 1 % (2 - 10 ml/l) at 40 - 50 °C</p> <p>For machine cleaning of flexible endoscopes: 0,5 % (5 ml/l)</p> <p>Cleaning manually: immersion bath: 0,5 – 2 %, 20 - 30 min ultrasonic bath: 0,5 – 2 %, 5 - 10 min</p> <p>Cleaner solution may optionally be heated to a maximum of 45 °C.</p> <p>The working solution of neodisher MediZym has to be replaced at least daily.</p> <p>Subsequently the neodisher MediZym-solution has to be rinsed out completely by sufficient rinsing. Do not mix with other products.</p> <p>Only for professional use.</p> |
| Technical data: | <p>Specific gravity (20 °C): 1.0 g/cm³</p> <p>pH-range (determined in deionised water, 20 °C) 0.2 - 2 %: 7.6 – 7.7</p> <p>Viscosity (Concentrate, 20 °C): < 10 mPas</p> |
| Ingredients: | <p>Ingredients according to Regulation (EC) No 648/2004 on detergents: < 5 %, anionic surfactants also enzymes, preservative (Phenoxyethanol, Methyl-,ethyl-,butyl-,propyl-,isopropylparaben)</p> |
| CE-marks: | neodisher MediZym conforms to the requirements of the legislation for medical devices according to the European directive 93/42/EC. |
| Storage information: | Store in a cool, frost-free place. Usable for 2 years when stored as recommended. |
| Hazard warnings and safety advice: | <p>neodisher MediZym has not to be classified according to the Dangerous Preparation Directive 1999/45/EC.</p> <p>Dispose only when container is empty and closed. For disposal of product residues, refer to Material Safety Data Sheet.</p> <p>For further safety information see EC-Safety data sheets. These are available e.g. under www.drweigert.de under the headline "Service".</p> |

MB 4040 GB/2-4
12/06



Chemische Fabrik Dr. Weigert GmbH & Co. KG
Mühlenhagen 85, D – 20539 Hamburg

Telefon: (040) 789 60 - 0
Telefax: (040) 789 60 - 120

E-Mail: info@drweigert.de
Internet: www.drweigert.de

Die Angaben dieses Merkblattes beruhen auf unseren derzeitigen Kenntnissen und Erfahrungen. Sie befreien den Verwender nicht von eigenen Prüfungen und Versuchen. Eine rechtlich verbindliche Zusicherung bestimmter Eigenschaften kann hieraus nicht abgeleitet werden.

With the above information, which is appropriate to our current knowledge we describe our product regarding possible safety necessities, but we do not involve any quality description or promise certain properties.