

neodisher[®] Mielclear

Liquid acidic rinsing agent
for use in dental washing machines



Main fields of application:	Rinsing agent for dental washing machines to prepare dental instruments, utensils and trays in dental practices, as well as in dental and orthodontic clinics.
Characteristics:	neodisher Mielclear achieves wetting of stainless steel surfaces, as well as plastics which are difficult to wet. The addition of acids permits the neutralisation of alkali residues and of alkaline waters.
Application and dosage:	Use in special washing machines: 0.2 - 0.4 ml/l automatically by suitable dosing devices. neodisher Mielclear is metered in the last rinsing stage. Do not mix with other products.
Technical data:	Specific gravity: (20 °C): 1.05 g/cm ³ pH-range (determined in deionised water, 20 °C) 0.2 - 0.4 ml/l: 4.1 - 3.5 Viscosity (concentrate, 20 °C): < 50 mPas
Ingredients:	Ingredients according to recommendation 89/542/EEC for detergents: < 5 % non-ionic surfactants
CE-marks:	neodisher Mielclear conforms to the requirements of the legislation for medical devices according to the European directive 93/42/EC.
Storage information:	The product is sensitive to frost. Usable for 3 years when stored as recommended.
Hazard warnings and safety advice:	Classification and labelling according to the Dangerous Preparation Directive 1999/45/EC for the concentrate as delivered. Hazard symbol C - corrosive contains: Phosphoric ester > 10 % Risk phrases: R 34 - Causes burns. Safety phrases: S 26 - In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S 27 - Take off immediately all contaminated clothing. S 28 - After contact with skin, wash immediately with plenty of water. S 36/37/39 - Wear suitable protective clothing, gloves and eye/face protection. S 45 - In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible). For further safety information see EC-Safety data sheets. These are available e.g. under www.drweigert.de under the headline "Service".

MB 4119 GB/2-1
03/03



Chemische Fabrik Dr. Weigert GmbH & Co. KG
Mühlenhagen 85, D - 20539 Hamburg

Telefon: (040) 789 60 - 0
Telefax: (040) 789 60 - 120

E-Mail: info@drweigert.de
Internet: www.drweigert.de

Die Angaben dieses Merkblattes beruhen auf unseren derzeitigen Kenntnissen und Erfahrungen. Sie befreien den Verwender nicht von eigenen Prüfungen und Versuchen. Eine rechtlich verbindliche Zusicherung bestimmter Eigenschaften kann hieraus nicht abgeleitet werden.

With the above information, which is appropriate to our current knowledge we describe our product regarding possible safety necessities, but we do not involve any quality description or promise certain properties.